


1 Login

You can

Login with your registered email


Or

Not yet an user?
You can sign up here for a new account with your email

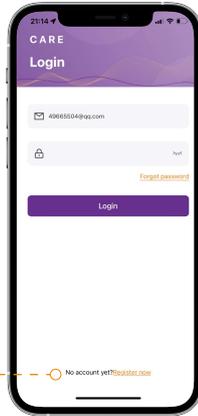

2 Home Screen

After login, you navigate to home screen, which displays a list of users you follow

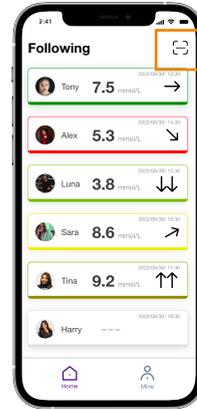


Each colored border represents different glucose status

- █ Normal
- █ Urgent glucose drop/rise
- █ Glucose variate abnormally

You can scan QR code from your patient's GN-Share app to add him/her

3 Patient Detail Screen

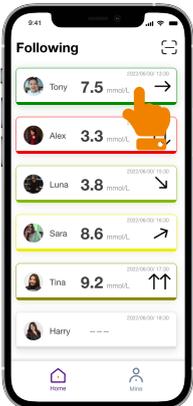

Tab on a patient from the list

You can navigate to detail screen of the selected patient



4 Patient Detail Screen Overview




Set glucose alert threshold

Turn on/off alert


Adjust alert threshold

Patient's information card

Show glucose data from past 4 hours

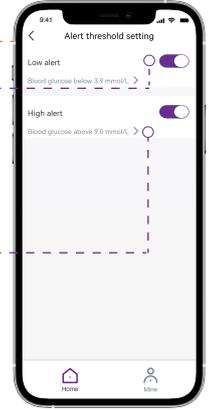

5 Personal Information Screen

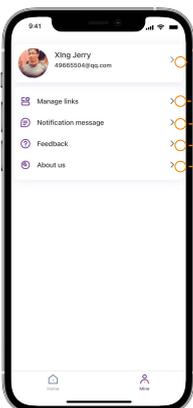

View and edit your personal information

Delete/Add contacts

View and manage messages

Share your suggestion and feedback about the app

View Term of use and Private policy




Note:

Not be used to make medical decisions


Information provided by GN-Carer should not be used or treat determine dosing. Please refer to the blood glucose monitoring system or consult a health-care professional.



Scan QR code to add a new patient

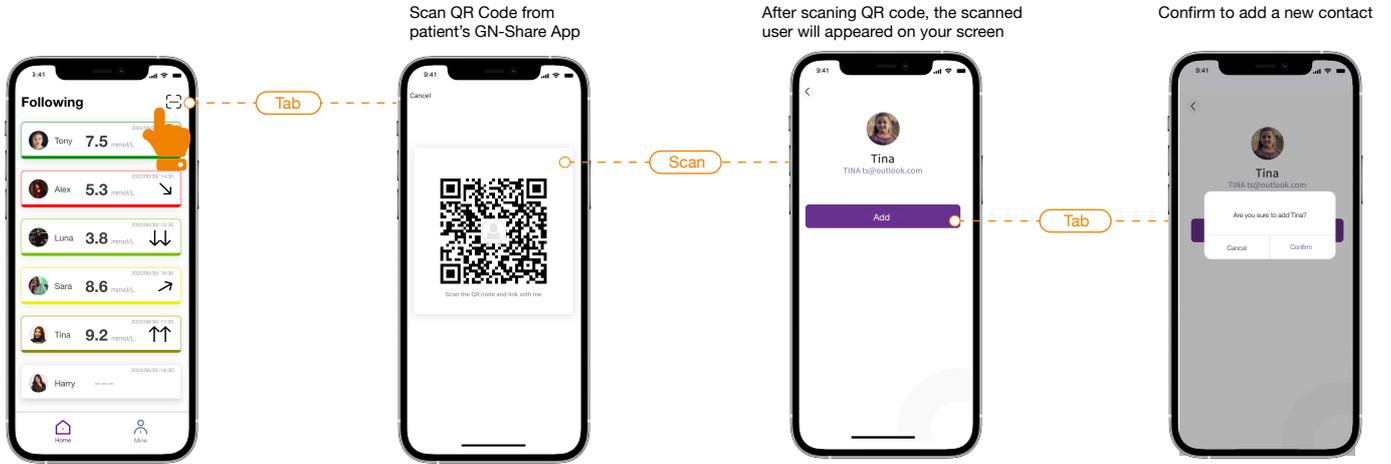

Manage Contacts

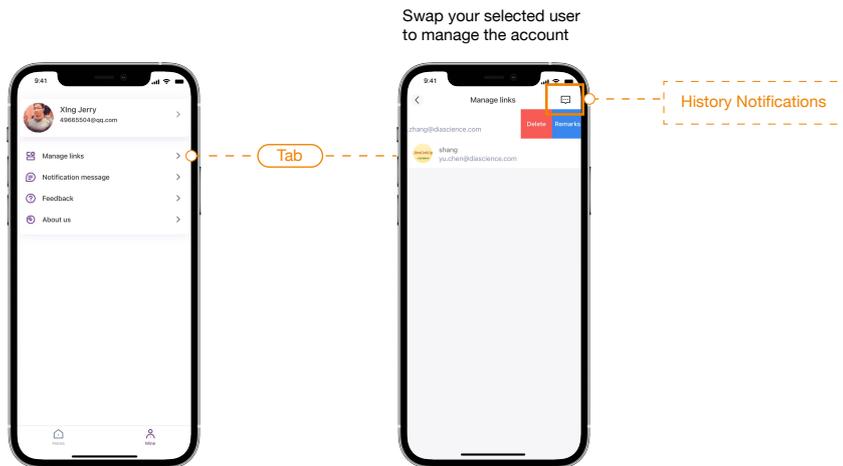

Read and Manage Messages

